



## Wound and eyewash in case of foreign objects


This wound and eyewash is a sterile 0.9 % sodium chloride solution that eases discomfort and prevents following complications caused by foreign bodies in the wounds or eyes.

### PRODUCT FACTS & BENEFITS

- Effectively removes dust, dirt, and contamination in the wounds or eyes
- Easy to handle - easily applied by the injured person or with assistance
- Flexible solutions - bottles for mobile use and stations for wallmounting
- Ergonomic eyecups providing a soft and pleasant flow
- Wide application - from industrial workplaces to laboratories and first aid kits
- High functional reliability
- The product is CE marked according to the Medical Device Regulation (EU) 2017/745
- Shelf life of 3.5 years



# Combine Plum Wound and Eyewash 0.9% Sodium Chloride with Plum Wound and Eyewash pH Neutral 4.9% Phosphate for workplaces with a risk of chemical accidents

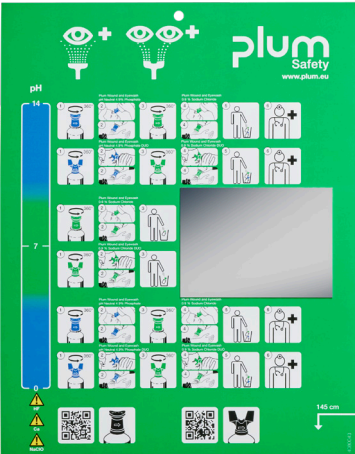

## Know your chemicals and ensure the best suited first aid

The cause of an accident determines which wound or eyewash is required. When an accident occurs, it is essential to determine whether it is caused by either a foreign object or by acids or alkali - and if possible, which pH value the substance in question holds.

When pH is either strongly acidic or alkaline, start rinsing with Plum Wound and Eyewash pH Neutral 4.9% Phosphate until the bottle is empty. Discard the bottle and rinse with Plum Wound and Eyewash 0.9% Sodium Chloride until you reach a doctor or the emergency room.

When pH is neutral, rinse with Plum Wound and Eyewash 0.9% Sodium Chloride until the bottle is empty and discard the bottle after use.

## ERGONOMIC EYECUPS

The bottles are equipped with a selection of ergonomic eyecups to ensure that the liquid can be applied correctly and in an even flow. Squeeze the bottle gently to generate a soft and pleasant flow.

### EYECUP




Flexible, ergonomic eyecup that keeps the eye lid open during use

### DUO EYECUP



Flexible, ergonomic eyecup that allows rinsing of both eyes at the same time

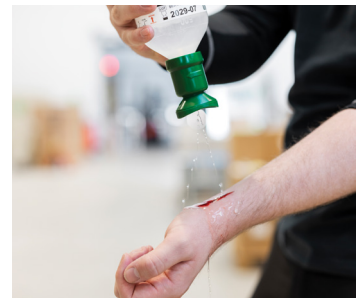

### Please note:

For wound irrigation, the shape of the eyecup is irrelevant.

## AVAILABLE SIZES

Art. no.	4691	4604	4707	4861	4800
Size	200 ml	500 ml	1000 ml	500 ml	1000 ml
Packaging	10 x 200 ml	12 x 500 ml	6 x 1000 ml	6 x 500 ml	6 x 1000 ml

info@plum.eu  
+45 6916 9600