




SAFETY AT WORK

How to place bottles

and why it is important always to turn the bottle upside down

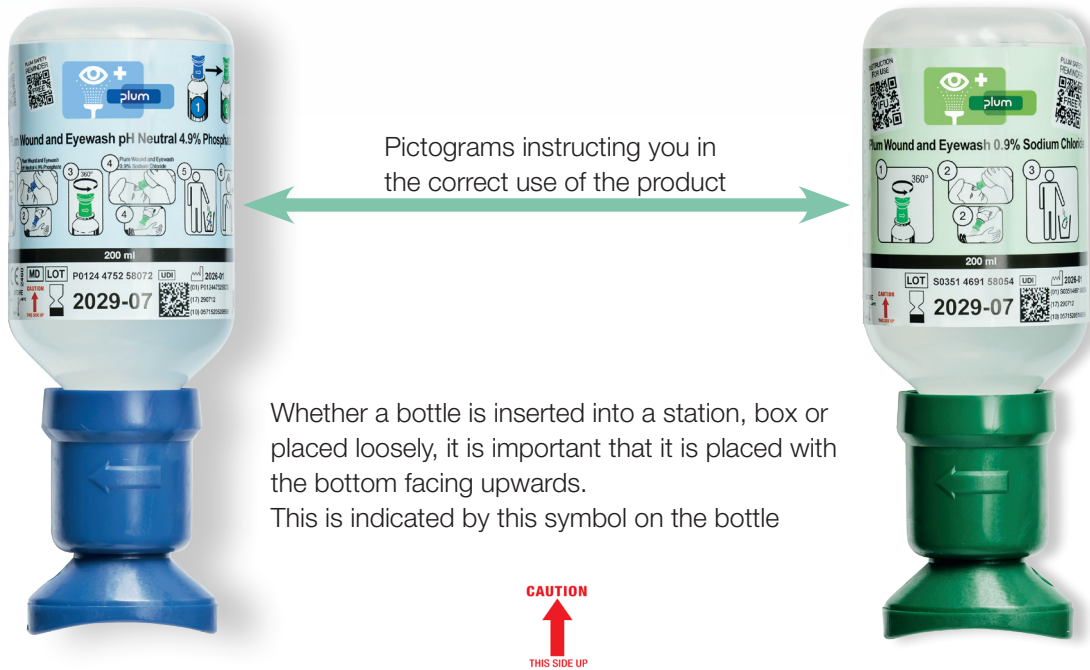


Whether a bottle is inserted into a station, box or placed loosely, it is important that it is placed with the bottom facing upwards. This is indicated by this symbol on the bottle

If you use the contents of a bottle and mistakenly save the bottle with any residual fluid, the contents will no longer be sterile. If the activated bottle is inserted correctly, it may leak a bit and you will realise that the bottle has already been opened. This avoids using liquid now being non-sterile and therefore should had been discarded.

Please always pay attention to pictograms and symbols on the bottle to ensure correct and safe use.

You can learn more about the meaning of symbols by using the QR code

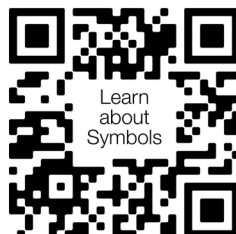

How to place bottles, when **not** using a box or station

Wall bracket - for 200 ml and 500 ml bottles - not suitable for DUO bottles



Green wall bracket

Packaging: 1 pcs

Article no.: 956430

Wall bracket for Plum Wound and Eyewash 0.9% Sodium Chloride 200 ml and 500 ml (art. no. 4691, 4604)



Blue wall bracket

Packaging: 1 pcs

Article no.: 956431

Wall bracket for Plum Wound and Eyewash pH Neutral 4.9% Phosphate 200 ml (art. no. 4752)

Wall hanger - for all 500 ml and 1000 ml bottles



Green wall hanger

Packaging: 1 pcs

Article no.: 956466

Wall hanger for Plum Wound and Eyewash 0.9% Sodium Chloride 500 ml and 1000 ml (art. no. 4604, 4707, 4861 and 4800)



Blue wall hanger

Packaging: 1 pcs

Article no.: 956407

Wall hanger for Plum Wound and Eyewash pH Neutral 4.9% Phosphate DUO 500 ml (art. no. 4801)

info@plum.eu
+45 6916 9600